
Back 2Sleep


INSTRUCTIONS FOR USE


NOTICE D'UTILISATION – GEBRAUCHSANWEISUNG
INSTRUCCIONES DE USO – ISTRUZIONI PER L'USO
INSTRUÇÕES DE USO – BRUGSANVISNING
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

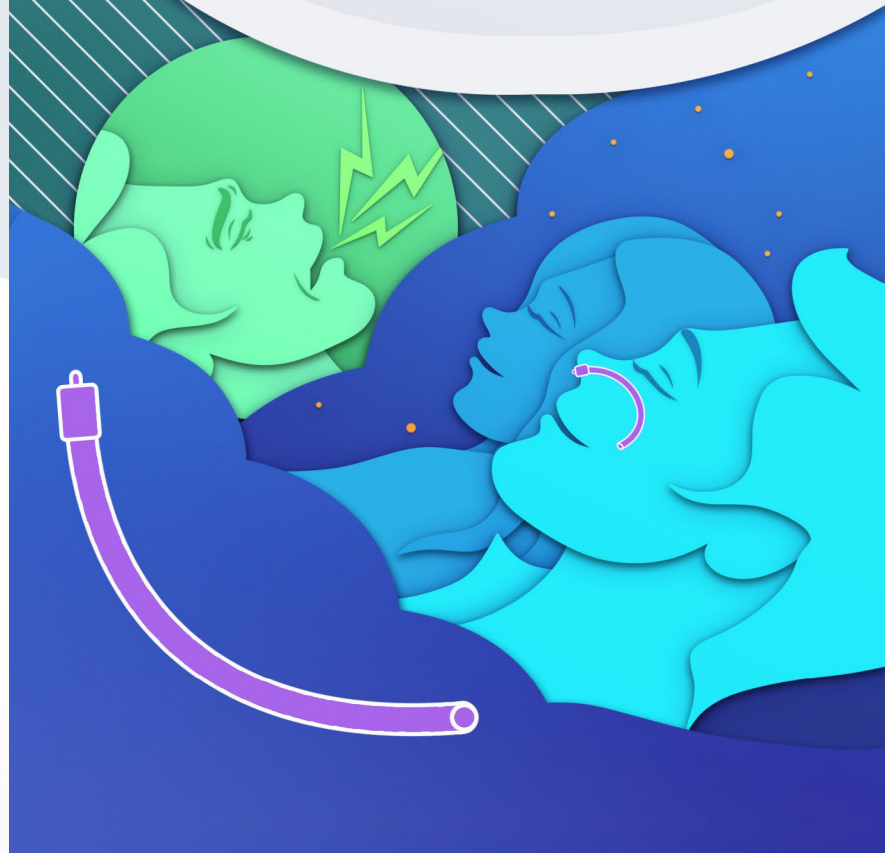→ Snorekit (A) to (F)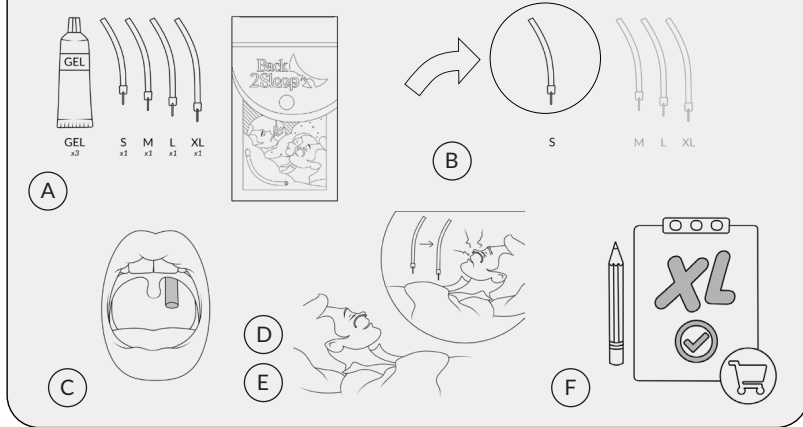

INSTRUCTIONS FOR USE (1) to (11)
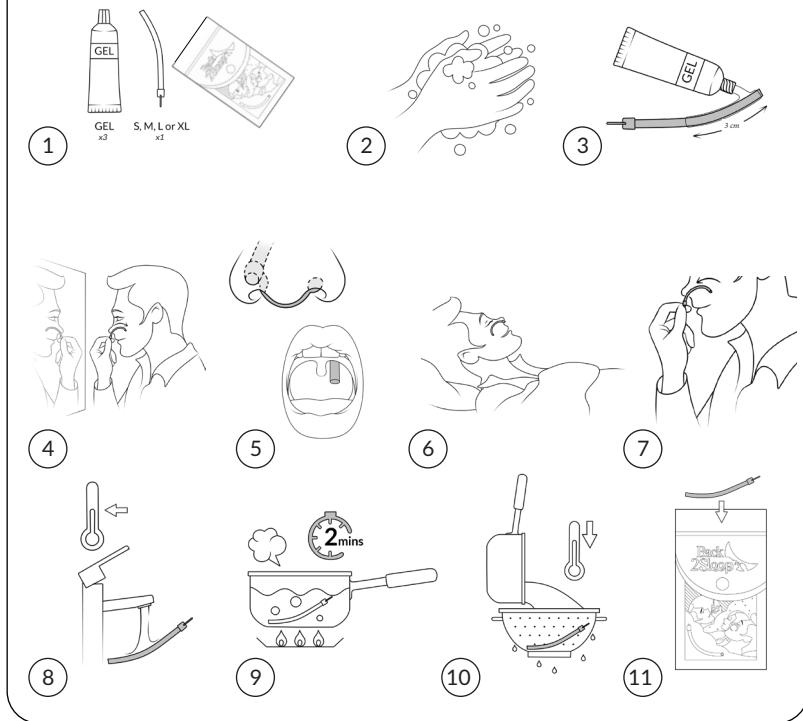
| | |
|---------------------------------|----|
| Notice d'utilisation..... | 2 |
| Instructions for use..... | 5 |
| Gebrauchsanweisung..... | 8 |
| Instrucciones de uso..... | 11 |
| Istruzioni per l'uso..... | 14 |
| Manual de instruções..... | 17 |
| Instrukcja dla użytkownika..... | 20 |
| Gebruiksaanwijzing..... | 23 |
| Οδηγίες χρήσης..... | 26 |

Back2Sleep - Tube nasal anti-ronflement

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Back2Sleep - Anti-snoring nasal tube

What is Back2Sleep used for?

Back2Sleep is a medical tube that helps you :

- Stop snoring
- Combat sleep apnea
- Breathe better at night
 - Sleep better

Which product do you have ?

| | |
|---|--|
| <p>BACK2SLEEP SNOREKIT - DISCOVERY KIT</p> <p>To find your ideal size</p> <ul style="list-style-type: none"> • 4 tubes in different sizes: S, M, L, and XL <ul style="list-style-type: none"> • 3 lubricating gel applicators • Allows you to test all sizes | <p>BACK2SLEEP S, M, L OR XL - MONTHLY PACK</p> <p>When you already know your size</p> <ul style="list-style-type: none"> • 1 tube of your chosen size (for 30 days) <ul style="list-style-type: none"> • 3 lubricating gel applicators |
|---|--|

DO YOU HAVE THE SNOREKIT ? START HERE TO FIND YOUR SIZE

Important : Just like shoes, everyone has their own size!

How to find the right size :

A/ Check that the packaging is not damaged.

B/ In the evening, before going to sleep, start by trying the smallest tube (size S).

C/ To check if the size is right :

- Open your mouth in front of a mirror
- The tip of the tube should extend at least 1 to 1.5 cm behind your palate below the uvula
- If you can't see clearly, try each size until you find the most effective (reduces snoring) and comfortable one

D/ If the tube feels uncomfortable or seems too short, try the next size up.

E/ Sleep with it in. If it's comfortable and effective, that's your size! If not, try a larger size the following night.

F/ Note your size (S, M, L, or XL) to order the corresponding monthly pack directly from your pharmacist, distributor, or online.

INSTRUCTIONS FOR USE - For all users

Before you begin :

1. Remove everything from the package.
2. Wash your hands thoroughly.

To insert the tube :

3. Apply a small amount of lubricating gel to the first 3 centimeters of the tube.

△ Do not put gel inside the tube!

4. Standing in front of a mirror, gently slide the tube into one nostril.

- The metal clip should be horizontal.

5. Check the position :

- Open your mouth in front of the mirror.
- You should see the tip of the tube protruding from the soft palate.

6. Go to bed.

The next morning :

7. Gently remove the tube.

8. Clean immediately :

- Rub by hand under warm water for 30 seconds until all mucus is removed.

9. Disinfection :

- 2 minutes in boiling water. The tube must be completely submerged.

10. Remove the tube with a strainer and allow to cool so as not to burn yourself.

11. Store in a clean, dry place.

12. In the evening, repeat from step 3.

△△ IMPORTANT PRECAUTIONS - MUST READ △△

Before you begin

- Consult your doctor before first use.
- Before using each tube for the first time, disinfect it according to steps 9 and 10 above.

Safety precautions

- Do not use if you are allergic to silicone.
- Maximum duration: 10 hours per night.
- Change the tube every 30 days after opening the package (even if little used)
- Use only one tube at a time (never two)
- If one nostril is blocked, use the other

It is normal if...

- You feel discomfort during the first 3-4 nights
- You feel like there is a foreign body in your nose

Stop immediately and consult a doctor in the following cases :

- Nosebleeds
- Difficulty breathing
- Frequent nausea
- Severe pain
- The tube changes color, smells bad, or is cracked
- The clip does not stay on your nose
- Accidental ingestion of the device

Size issues? Here are the signs :

Tube too short :

- Significant discomfort and real inconvenience
- The uvula sticks to the tube
- The tube is pushed out by the tongue, causing obstruction at the base of the tongue
- Mucus frequently blocks the tube
- Ineffective against snoring

→ Choose a larger size

Tube too long :

- Significant nausea
- Discomfort in the epiglottis

→ Choose a shorter size

Not suitable for :

- × Children
- × Individuals with very restless sleep
- × Unconscious individuals
- × Individuals under guardianship (except under supervision)

Consult your doctor first if you have :

- Breathing problems
- Nose or throat disorders
- Swollen tonsils
- Gastrointestinal problems
- Injuries to the nose

Possible side effects :

May occur but usually disappear :

- Difficulty inserting/removing the tube
- Foreign body sensation
- Temporary dry cough
- The tube may move or come out during the night
- Rarely: silicone allergy

Handling precautions :

- Do not bend the metal clip
- Do not detach the clip from the tube
- Do not stretch the silicone tube
- Never dismantle the device
- △ Only use the gel provided
- Be careful not to burn yourself with boiling water

Storage

- Store in a clean, dry place
- Between 4°C and 40°C
- Away from sunlight
- Out of reach of children
- Check the expiry date (YYMMDD format)

In case of serious problems

Contact Back2Sleep: www.back2sleep.eu

Manufacturer

Back2Sleep™ - 188 rue de Rivoli - 75001 - Paris - France

🗨️ Tip: Once you have found your size with the SNOREKIT, order your size (S, M, L, or XL) directly from your pharmacist, distributor, or at www.back2sleep.eu

Back2Sleep – Anti-Schnarch-Nasenschlauch

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